


Oro Tox

By Robert P. McBride DDS, Inc.



Use the enclosed color scale with good, indirect lighting (avoid reflections). Move the cuvette over the scale cutouts until ink coverage. Clearly positive, clinically significant results are easier to read. Read the number (toxin value) and in the second part the letters (protein toxin and inflammation protein value) and note.

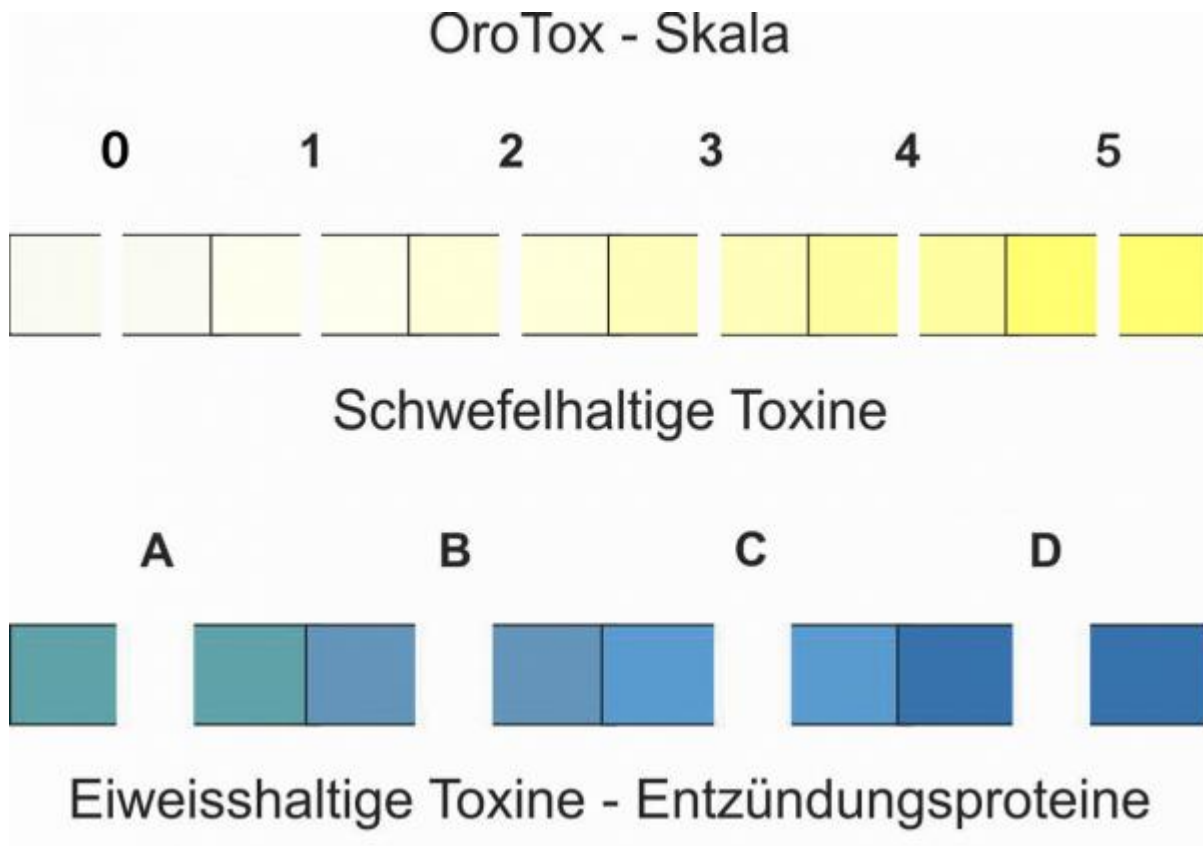

Meaning of the measured values - sulfur toxins

- 0 = No proof
- 1 = Easy
- 2 = Mild
- 3 = Clear
- 4 = Strong
- 5 = Extremely strong

Meaning of the measured values - protein toxin and inflammation protein value.

Copyright © 2019 Robert P. McBride DDS, Inc. All Rights Reserved.